ZOLL®

AutoPulse® Quick Case

Instruction Manual

9650-000850-40 Rev B

GENERAL INFORMATION

The AutoPulse Quick Case is exclusively for use with the AutoPulse® Resuscitation System Model 100.

The AutoPulse Quick Case is designed to both protect the AutoPulse during storage and to as be used as a transportation device when the AutoPulse is in use.

This instruction manual is intended to provide users with information on how to utilize the AutoPulse Quick Case. Please refer to the AutoPulse Resuscitation System Model 100 User Guide for operating steps and maintenance requirements for the AutoPulse.

MAINTENANCE

The AutoPulse Quick Case can be hand-washed by cleaning agents and machined washed utilizing light-duty detergent and water at a maximum temperature of 86° F (30° C). Hang-drying the AutoPulse Quick Case is recommended to avoid damage to the waterproof coating.

Prior to use, always check the AutoPulse Quick Case for any damage.

Explanation of Used Symbols	
	Read User Guide
	CE mark

Open Layout Dimensions	
Height	8 ft. 5 inches (253 cm)
Width	3 ft. 6 inches (109 cm)
Weight	7.9 lbs. (3600 g)

Closed Layout Dimensions with AutoPulse	
Height	2 ft. 9 inches (84 cm)
Width	1 ft. 4 inches (40 cm)
Depth	6 inches (15 cm)

LAYOUT

OUTSIDE	
1	Positioning symbol
2	Window for AutoPulse display
3	Cut-out for On/Off button
4	Protection strap on head side
5	Handle (8x)
6	Case for trauma shears
7	Shoulder belt
8	Handle to position the AutoPulse
9	Sheet for carrying the patient

INSIDE	
10	Case for cable strap
11	Case for extra harness
12	Flap for AutoPulse fixation
13	Waist strap for patient
14	Flap for patient's feet

FOLDING INSTRUCTIONS

Attachment and folding instructions for the Quick Case on the AutoPulse

1

Fully open the Quick Case

Make sure that the Velcro on the bag and on the bottom side of the AutoPulse is in good condition.

2

Place the AutoPulse into the holding strap

Make sure that the AutoPulse fits tightly.

3

Place the AutoPulse LifeBand® and patient waist strap on the AutoPulse platform

Make sure that the position of both LifeBand and waist strap is correct.

4

Pull the gray part of the Quick Case from the bottom to the top

FOLDING INSTRUCTIONS *(continued)*

5

Fold the gray part of the Quick Case in half, from the top to the bottom

6

Continue folding the gray part of the Quick Case in half again from the top to the bottom

7

Fold left side inwards

8

Fold right side inwards
Secure the four black straps by connecting the buckles and tightening the straps.

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